

Urgent Field Safety Notice

PST 500

FA Number: FA-2024-042

Manufacturer: Baxter Medical Systems GmbH + Co. KG (Single Registration Number: DE-MF-000005071)

Correction

June 2024

Dear Sir, Madam,

Baxter Healthcare Corporation is issuing a Correction for the PST 500 U Precision Surgical Table due to a potential issue with loose spring pins in the spindle drives. This can cause the tabletop to unexpectedly tilt or move at any time, even without active use of the surgical table. This could potentially result in unintentional patient movement during surgical procedures and/or preparation for surgical procedures, including transport.

Baxter will correct this issue by providing a service kit that can be installed either by a hospital technician or by a Baxter technician or authorized representative. If the service kit is self-installed, a Baxter technician or authorized representative will follow up to verify proper installation.

Affected Product

Product Code	Product Name	Serial Number	UDI Number
4080100	PST 500	Refer to Attachment A	00887761973466

Hazard Involved

A sudden drop and/or tilt of the surgical tabletop during a surgical procedure may cause unintended patient movement or fall. This could lead to delay or interruption in therapy, ineffective CPR, and could cause critical patient harm such as major musculoskeletal or vital organ injuries. Baxter has received six complaints related to this issue; however, no serious injury was reported.

Actions to be Taken by Customers

1. Immediately cease all use of the impacted units until a correction has been completed. If you have alternative tables available, please use them until the service kit is installed by the hospital technician, a Baxter technician, or an authorized representative.
2. Below are two options for correction of the impacted units:
 - a. Self-installation of the service kit (part number 2086594) executed by a hospital-defined technician per the technical service bulletin included with the service kit. Once the self-installation is successful, the impacted units are ready for use. For any questions regarding the installation of the service kit, please contact Baxter Technical Support.

3. A Baxter representative will contact you to schedule inspection of self-corrected units or correct the impacted units at your facility. Do not use the impacted units until the correction has been executed. To request correction of your impacted tables, contact Baxter Technical Support.
4. Complete the enclosed customer reply form and return it to Baxter by either faxing it or scanning and e-mailing it or sending it by post, even if you don't have any inventory. Returning the customer reply form promptly will confirm your receipt of this notification and prevent you from receiving repeat notices.
5. Please provide this information to all users of the PST 500 Precision Surgical Table. If you distributed this product to other facilities or departments within your institution, please forward a copy of this communication to them.
6. If you purchased this product from a distributor, please note that the Baxter customer reply form is not applicable. If a reply form is provided by your distributor or wholesaler, please return it to the supplier according to their instructions.
7. If you are a dealer, wholesaler, distributor/reseller, or original equipment manufacturer (OEM) that distributed any affected product to other facilities, please notify your customers of this Correction in accordance with your customary procedures.

Further Information and Support

For general questions regarding this communication or any product issue you are experiencing, contact Baxter.

The local Ministry of Health (MOH) has been notified of this action.

We apologize for any inconvenience this may cause you and your staff.

Sincerely,

Baxter Healthcare Corporation

Enclosure: Baxter Customer Reply Form

Attachment A: Affected Product Table



(Customer communication)

CUSTOMER REPLY FORM related to Product Correction letter dated XXXXXX (to be completed locally)

Product Name: PST 500 Surgical Table

Product code: 4080100

Serial Number: (to be adapted locally)

Please complete and return one copy of this form per facility either by fax (Fax : _____) or by e-mail (_____) as confirmation that you have received this notification. A fax cover sheet is not required. (Can be adapted locally).

Facility Name and Address:	
Reply Confirmation Completed By (Please Print):	
Title (Please print):	
Email and/or Telephone Number (including Area Code):	

Please list the specific products and serial numbers in your facility below*:

Product Code	Serial number

*You may attach an additional sheet if required.

Your signature below indicates that you have received the attached letter; performed the actions as outlined in the letter as needed; and disseminated this information to staff and other services or facilities as applicable.

Signature/Date: REQUIRED FIELD	_____
----------------------------------------------	-------

Attachment A: Affected Product Table

Product Code	Device Name	Serial Number
4080100	PST 500	108463472
4080100	PST 500	108088844
4080100	PST 500	108275110
4080100	PST 500	108339159
4080100	PST 500	108339160
4080100	PST 500	108364926
4080100	PST 500	108486930
4080100	PST 500	108842441
4080100	PST 500	109031472
4080100	PST 500	109097327
4080100	PST 500	109315046
4080100	PST 500	109315053
4080100	PST 500	109315056
4080100	PST 500	109315061
4080100	PST 500	109315062
4080100	PST 500	108865824
4080100	PST 500	109309750
4080100	PST 500	109311005
4080100	PST 500	109311031
4080100	PST 500	109311035
4080100	PST 500	109311036
4080100	PST 500	109311040
4080100	PST 500	109311046
4080100	PST 500	109311134
4080100	PST 500	109390920
4080100	PST 500	109390921
4080100	PST 500	109390922
4080100	PST 500	109390923
4080100	PST 500	109405715
4080100	PST 500	109405718
4080100	PST 500	109405719
4080100	PST 500	109405720
4080100	PST 500	109405721
4080100	PST 500	109405723
4080100	PST 500	109451050
4080100	PST 500	109451051
4080100	PST 500	109451052
4080100	PST 500	109451053
4080100	PST 500	108570641

Attachment A: Affected Product Table

4080100	PST 500	109087820
4080100	PST 500	109087835
4080100	PST 500	109087840
4080100	PST 500	109147934
4080100	PST 500	109148117
4080100	PST 500	109148122
4080100	PST 500	109148123
4080100	PST 500	109213582
4080100	PST 500	109299611
4080100	PST 500	107933120
4080100	PST 500	107935068
4080100	PST 500	107936297
4080100	PST 500	108133275
4080100	PST 500	108160146
4080100	PST 500	108168346
4080100	PST 500	108168347
4080100	PST 500	108174395
4080100	PST 500	108174576
4080100	PST 500	108233135
4080100	PST 500	108318588
4080100	PST 500	108338479
4080100	PST 500	108370011
4080100	PST 500	108370158
4080100	PST 500	108438646
4080100	PST 500	108444090
4080100	PST 500	108492786
4080100	PST 500	108514176
4080100	PST 500	108547192
4080100	PST 500	108616027
4080100	PST 500	108697596
4080100	PST 500	108697597
4080100	PST 500	108770713
4080100	PST 500	108770714
4080100	PST 500	108770715
4080100	PST 500	108770716
4080100	PST 500	108781166
4080100	PST 500	108781167
4080100	PST 500	108781171
4080100	PST 500	108781172
4080100	PST 500	108865835
4080100	PST 500	108922932

Attachment A: Affected Product Table

4080100	PST 500	108922933
4080100	PST 500	108922934
4080100	PST 500	109030478
4080100	PST 500	109299628
4080100	PST 500	109456525
4080100	PST 500	109477670
4080100	PST 500	108153368
4080100	PST 500	108233131
4080100	PST 500	108233132
4080100	PST 500	108318601
4080100	PST 500	108318602
4080100	PST 500	108339161
4080100	PST 500	108392476
4080100	PST 500	108547187
4080100	PST 500	108547190
4080100	PST 500	108616036
4080100	PST 500	108616037
4080100	PST 500	108616038
4080100	PST 500	108683181
4080100	PST 500	108716145
4080100	PST 500	108815855
4080100	PST 500	108815856
4080100	PST 500	108815868
4080100	PST 500	108815869
4080100	PST 500	108850243
4080100	PST 500	108884051
4080100	PST 500	108884059
4080100	PST 500	108893834
4080100	PST 500	109061360
4080100	PST 500	109061363
4080100	PST 500	109061365
4080100	PST 500	109061366
4080100	PST 500	109087822
4080100	PST 500	109148168
4080100	PST 500	109148169
4080100	PST 500	109213579
4080100	PST 500	109229486
4080100	PST 500	109229487
4080100	PST 500	109367376
4080100	PST 500	109367380
4080100	PST 500	109375647

Attachment A: Affected Product Table

4080100	PST 500	109456167
4080100	PST 500	109486033
4080100	PST 500	109512030
4080100	PST 500	108062348
4080100	PST 500	108062349
4080100	PST 500	108062351
4080100	PST 500	108062357
4080100	PST 500	108088843
4080100	PST 500	108153374
4080100	PST 500	108153375
4080100	PST 500	108153376
4080100	PST 500	108153377
4080100	PST 500	108153381
4080100	PST 500	108153382
4080100	PST 500	108877080
4080100	PST 500	108877081
4080100	PST 500	109072296
4080100	PST 500	109072300
4080100	PST 500	109072301
4080100	PST 500	109072307
4080100	PST 500	109113382
4080100	PST 500	108650853
4080100	PST 500	108650860
4080100	PST 500	108650865
4080100	PST 500	108650866
4080100	PST 500	108310307
4080100	PST 500	108318603
4080100	PST 500	108339134
4080100	PST 500	108381555
4080100	PST 500	108396851
4080100	PST 500	108486931
4080100	PST 500	108486933
4080100	PST 500	108492527
4080100	PST 500	108570463
4080100	PST 500	108570464
4080100	PST 500	108570612
4080100	PST 500	108570613
4080100	PST 500	108570614
4080100	PST 500	108570615
4080100	PST 500	108570616
4080100	PST 500	108570619

Attachment A: Affected Product Table

4080100	PST 500	108616025
4080100	PST 500	108616026
4080100	PST 500	108649052
4080100	PST 500	108649054
4080100	PST 500	108649055
4080100	PST 500	108650848
4080100	PST 500	108658997
4080100	PST 500	108659406
4080100	PST 500	108659407
4080100	PST 500	108659408
4080100	PST 500	108659409
4080100	PST 500	108659410
4080100	PST 500	108659411
4080100	PST 500	108781164
4080100	PST 500	108884065
4080100	PST 500	108884069
4080100	PST 500	108884076
4080100	PST 500	109023846
4080100	PST 500	109031471
4080100	PST 500	109097317
4080100	PST 500	109097318
4080100	PST 500	109097319
4080100	PST 500	109097320
4080100	PST 500	109097325
4080100	PST 500	109097326
4080100	PST 500	109171333
4080100	PST 500	109171334
4080100	PST 500	109171335
4080100	PST 500	109180605
4080100	PST 500	109180611
4080100	PST 500	109180612
4080100	PST 500	109180613
4080100	PST 500	109180614
4080100	PST 500	109181281
4080100	PST 500	109181282
4080100	PST 500	109181283
4080100	PST 500	109200077
4080100	PST 500	109200081
4080100	PST 500	109200082
4080100	PST 500	109200083
4080100	PST 500	109243539

Attachment A: Affected Product Table

4080100	PST 500	109243540
4080100	PST 500	109243541
4080100	PST 500	109243542
4080100	PST 500	109243543
4080100	PST 500	109243544
4080100	PST 500	109243545
4080100	PST 500	109243546
4080100	PST 500	109243571
4080100	PST 500	109243573
4080100	PST 500	109243589
4080100	PST 500	109254345
4080100	PST 500	109254346
4080100	PST 500	109254347
4080100	PST 500	109254353
4080100	PST 500	109254354
4080100	PST 500	109370580
4080100	PST 500	109397520
4080100	PST 500	109400209
4080100	PST 500	109400210
4080100	PST 500	109400211
4080100	PST 500	109400214
4080100	PST 500	109400215
4080100	PST 500	109400216
4080100	PST 500	109400223
4080100	PST 500	109400225
4080100	PST 500	109401139
4080100	PST 500	109401140
4080100	PST 500	109401175
4080100	PST 500	109401186
4080100	PST 500	109401220
4080100	PST 500	109401221
4080100	PST 500	109401222
4080100	PST 500	109401223
4080100	PST 500	109401227
4080100	PST 500	109401228
4080100	PST 500	109401270
4080100	PST 500	109401275
4080100	PST 500	109410582
4080100	PST 500	109410583
4080100	PST 500	109410672
4080100	PST 500	109410737

Attachment A: Affected Product Table

4080100	PST 500	109410739
4080100	PST 500	109414016
4080100	PST 500	109414019
4080100	PST 500	109414080
4080100	PST 500	109414153
4080100	PST 500	109414154
4080100	PST 500	109414160
4080100	PST 500	109414161
4080100	PST 500	109414169
4080100	PST 500	109414173
4080100	PST 500	109414175
4080100	PST 500	109421739
4080100	PST 500	109421740
4080100	PST 500	109421741
4080100	PST 500	109421749
4080100	PST 500	109421750
4080100	PST 500	109424506
4080100	PST 500	109424508
4080100	PST 500	109424509
4080100	PST 500	109424510
4080100	PST 500	109424512
4080100	PST 500	109447780
4080100	PST 500	109447784
4080100	PST 500	109447789
4080100	PST 500	109447790
4080100	PST 500	109451049
4080100	PST 500	109458851
4080100	PST 500	109458877
4080100	PST 500	109458879
4080100	PST 500	109514916
4080100	PST 500	108153369
4080100	PST 500	108339170
4080100	PST 500	108391590
4080100	PST 500	108391591
4080100	PST 500	108805465
4080100	PST 500	108805466
4080100	PST 500	108805467
4080100	PST 500	108805468
4080100	PST 500	108805533
4080100	PST 500	108805534
4080100	PST 500	108805595

Attachment A: Affected Product Table

4080100	PST 500	109147940
4080100	PST 500	109218613
4080100	PST 500	109218614
4080100	PST 500	109218615
4080100	PST 500	109218616
4080100	PST 500	109218617
4080100	PST 500	109445769
4080100	PST 500	109486035
4080100	PST 500	108088840
4080100	PST 500	108088841
4080100	PST 500	108160149
4080100	PST 500	108294078
4080100	PST 500	108294106
4080100	PST 500	108338488
4080100	PST 500	108339169
4080100	PST 500	108396857
4080100	PST 500	108396858
4080100	PST 500	108418764
4080100	PST 500	108448305
4080100	PST 500	108477229
4080100	PST 500	108492526
4080100	PST 500	108507954
4080100	PST 500	108507955
4080100	PST 500	108507956
4080100	PST 500	108508003
4080100	PST 500	108597115
4080100	PST 500	108597116
4080100	PST 500	108597117
4080100	PST 500	108597118
4080100	PST 500	108597119
4080100	PST 500	108631111
4080100	PST 500	108631114
4080100	PST 500	108805393
4080100	PST 500	108805464
4080100	PST 500	108850223
4080100	PST 500	108865836
4080100	PST 500	108903851
4080100	PST 500	109004473
4080100	PST 500	109004474
4080100	PST 500	109004475
4080100	PST 500	109004476

Attachment A: Affected Product Table

4080100	PST 500	109113378
4080100	PST 500	109147930
4080100	PST 500	109147931
4080100	PST 500	109147933
4080100	PST 500	109147935
4080100	PST 500	109147936
4080100	PST 500	109147937
4080100	PST 500	109147938
4080100	PST 500	109147939
4080100	PST 500	109148170
4080100	PST 500	109148171
4080100	PST 500	109167191
4080100	PST 500	109167194
4080100	PST 500	109218585
4080100	PST 500	109218589
4080100	PST 500	109229485
4080100	PST 500	109233198
4080100	PST 500	109233199
4080100	PST 500	109233200
4080100	PST 500	109243538
4080100	PST 500	109279937
4080100	PST 500	109315023
4080100	PST 500	109315024
4080100	PST 500	109328443
4080100	PST 500	109409167
4080100	PST 500	109409230
4080100	PST 500	109409293
4080100	PST 500	109409295
4080100	PST 500	109409296
4080100	PST 500	109456134
4080100	PST 500	109465611
4080100	PST 500	109465766
4080100	PST 500	109465767
4080100	PST 500	108201335
4080100	PST 500	108201338
4080100	PST 500	108201344
4080100	PST 500	108201349
4080100	PST 500	108201407
4080100	PST 500	108201419
4080100	PST 500	108201420
4080100	PST 500	108201422

Attachment A: Affected Product Table

4080100	PST 500	108201424
4080100	PST 500	108201425
4080100	PST 500	108201427
4080100	PST 500	108230967
4080100	PST 500	108233137
4080100	PST 500	108233138
4080100	PST 500	108233140
4080100	PST 500	108233142
4080100	PST 500	108233143
4080100	PST 500	108233144
4080100	PST 500	108233145
4080100	PST 500	108241923
4080100	PST 500	108241926
4080100	PST 500	108241927
4080100	PST 500	108241928
4080100	PST 500	108241929
4080100	PST 500	108241930
4080100	PST 500	108241931
4080100	PST 500	108253415
4080100	PST 500	108253416
4080100	PST 500	108288929
4080100	PST 500	108288934
4080100	PST 500	108288940
4080100	PST 500	108288945
4080100	PST 500	108288946
4080100	PST 500	108294091
4080100	PST 500	108294092
4080100	PST 500	108294093
4080100	PST 500	108294094
4080100	PST 500	108352835
4080100	PST 500	108352839
4080100	PST 500	108352840
4080100	PST 500	108352841
4080100	PST 500	108352842
4080100	PST 500	108352843
4080100	PST 500	108352844
4080100	PST 500	108352845
4080100	PST 500	108352849
4080100	PST 500	108352850
4080100	PST 500	108352851
4080100	PST 500	108352852

Attachment A: Affected Product Table

4080100	PST 500	108352855
4080100	PST 500	108352856
4080100	PST 500	108352857
4080100	PST 500	108369985
4080100	PST 500	108369986
4080100	PST 500	108369987
4080100	PST 500	108370007
4080100	PST 500	108370008
4080100	PST 500	108370009
4080100	PST 500	108370010
4080100	PST 500	108459434
4080100	PST 500	108469426
4080100	PST 500	108469427
4080100	PST 500	108469428
4080100	PST 500	108469430
4080100	PST 500	108477224
4080100	PST 500	108477225
4080100	PST 500	108477226
4080100	PST 500	108477227
4080100	PST 500	108477228
4080100	PST 500	108508696
4080100	PST 500	108508697
4080100	PST 500	108547193
4080100	PST 500	108570624
4080100	PST 500	108570635
4080100	PST 500	108577055
4080100	PST 500	108577058
4080100	PST 500	108577059
4080100	PST 500	108577060
4080100	PST 500	108577061
4080100	PST 500	108577062
4080100	PST 500	108577063
4080100	PST 500	108577072
4080100	PST 500	108701337
4080100	PST 500	108701340
4080100	PST 500	108701341
4080100	PST 500	108701348
4080100	PST 500	108701350
4080100	PST 500	108701351
4080100	PST 500	108701352
4080100	PST 500	108701353

Attachment A: Affected Product Table

4080100	PST 500	108716111
4080100	PST 500	108716112
4080100	PST 500	108716113
4080100	PST 500	108716114
4080100	PST 500	108716115
4080100	PST 500	108716116
4080100	PST 500	108716117
4080100	PST 500	108716119
4080100	PST 500	108716121
4080100	PST 500	108716123
4080100	PST 500	108741634
4080100	PST 500	108741635
4080100	PST 500	108741636
4080100	PST 500	108741637
4080100	PST 500	108741638
4080100	PST 500	108741642
4080100	PST 500	108741643
4080100	PST 500	108741644
4080100	PST 500	108741645
4080100	PST 500	108741646
4080100	PST 500	108741647
4080100	PST 500	108741648
4080100	PST 500	108741651
4080100	PST 500	108741652
4080100	PST 500	108741687
4080100	PST 500	108741894
4080100	PST 500	108741895
4080100	PST 500	108741902
4080100	PST 500	108741903
4080100	PST 500	108741904
4080100	PST 500	108741915
4080100	PST 500	108741917
4080100	PST 500	108741918
4080100	PST 500	108741919
4080100	PST 500	108741920
4080100	PST 500	108741921
4080100	PST 500	108741922
4080100	PST 500	108754143
4080100	PST 500	108754153
4080100	PST 500	108754154
4080100	PST 500	108754155

Attachment A: Affected Product Table

4080100	PST 500	108754156
4080100	PST 500	108754158
4080100	PST 500	108754159
4080100	PST 500	108754160
4080100	PST 500	108754161
4080100	PST 500	108754162
4080100	PST 500	108776352
4080100	PST 500	108776353
4080100	PST 500	108780598
4080100	PST 500	108825614
4080100	PST 500	108825617
4080100	PST 500	108825619
4080100	PST 500	108833940
4080100	PST 500	108833980
4080100	PST 500	108883936
4080100	PST 500	108883941
4080100	PST 500	109083526
4080100	PST 500	109083527
4080100	PST 500	109330129
4080100	PST 500	109330130
4080100	PST 500	109330131
4080100	PST 500	109330133
4080100	PST 500	109330134
4080100	PST 500	109330135
4080100	PST 500	109330136
4080100	PST 500	109330138
4080100	PST 500	109330139
4080100	PST 500	109330140
4080100	PST 500	109349068
4080100	PST 500	109349069
4080100	PST 500	109349070
4080100	PST 500	109349071
4080100	PST 500	109375648
4080100	PST 500	109481159
4080100	PST 500	109493582
4080100	PST 500	109494578
4080100	PST 500	108683182
4080100	PST 500	108683202
4080100	PST 500	108716185
4080100	PST 500	108716186
4080100	PST 500	108716191

Attachment A: Affected Product Table

4080100	PST 500	108716193
4080100	PST 500	108727584
4080100	PST 500	108727585
4080100	PST 500	108770722
4080100	PST 500	108815876
4080100	PST 500	108884060
4080100	PST 500	108884062
4080100	PST 500	108884064
4080100	PST 500	108977042
4080100	PST 500	109083522
4080100	PST 500	109083523
4080100	PST 500	109087872
4080100	PST 500	109087873
4080100	PST 500	109087877
4080100	PST 500	109087878
4080100	PST 500	109478600
4080100	PST 500	109478602
4080100	PST 500	109478603
4080100	PST 500	109478604
4080100	PST 500	108922936
4080100	PST 500	109254348
4080100	PST 500	109254349
4080100	PST 500	109465609
4080100	PST 500	109465610
4080100	PST 500	108318600
4080100	PST 500	109502170
4080100	PST 500	109502272
4080100	PST 500	108267733
4080100	PST 500	108757446
4080100	PST 500	108815875
4080100	PST 500	108815877
4080100	PST 500	109328458
4080100	PST 500	109494570
4080100	PST 500	108338480
4080100	PST 500	109364907
4080100	PST 500	108781165
4080100	PST 500	108160150
4080100	PST 500	108318599
4080100	PST 500	108338471
4080100	PST 500	108338472
4080100	PST 500	108597120

Attachment A: Affected Product Table

4080100	PST 500	108664834
4080100	PST 500	108664835
4080100	PST 500	108664842
4080100	PST 500	108664843
4080100	PST 500	108664844
4080100	PST 500	108685872
4080100	PST 500	108716134
4080100	PST 500	108716150
4080100	PST 500	108716153
4080100	PST 500	108741639
4080100	PST 500	108741640
4080100	PST 500	108741641
4080100	PST 500	108815871
4080100	PST 500	108815872
4080100	PST 500	108815873
4080100	PST 500	108833983
4080100	PST 500	108842442
4080100	PST 500	108857271
4080100	PST 500	108857272
4080100	PST 500	109030701
4080100	PST 500	109031468
4080100	PST 500	109031469
4080100	PST 500	109097328
4080100	PST 500	109109780
4080100	PST 500	109148118
4080100	PST 500	109148120
4080100	PST 500	109148141
4080100	PST 500	109148142
4080100	PST 500	109171240
4080100	PST 500	109171241
4080100	PST 500	109171302
4080100	PST 500	109213574
4080100	PST 500	109218590
4080100	PST 500	109218592
4080100	PST 500	109321529
4080100	PST 500	109381329
4080100	PST 500	109381330
4080100	PST 500	109409294
4080100	PST 500	109427315
4080100	PST 500	109427317
4080100	PST 500	109427318

Attachment A: Affected Product Table

4080100	PST 500	109427320
4080100	PST 500	109427321
4080100	PST 500	109445770
4080100	PST 500	109460966
4080100	PST 500	109460967
4080100	PST 500	109462855
4080100	PST 500	109464393
4080100	PST 500	109464394
4080100	PST 500	109464395
4080100	PST 500	109464396
4080100	PST 500	109464397
4080100	PST 500	109496263
4080100	PST 500	109496289
4080100	PST 500	109496290
4080100	PST 500	109496291
4080100	PST 500	109496292
4080100	PST 500	109506784
4080100	PST 500	109506786
4080100	PST 500	109506828
4080100	PST 500	108174610
4080100	PST 500	108253418
4080100	PST 500	108781142
4080100	PST 500	109041066
4080100	PST 500	108381553
4080100	PST 500	108381554
4080100	PST 500	108389524
4080100	PST 500	108389525
4080100	PST 500	108389526
4080100	PST 500	108389527
4080100	PST 500	108391587
4080100	PST 500	108518856
4080100	PST 500	108518859
4080100	PST 500	108518860
4080100	PST 500	108518863
4080100	PST 500	108518864
4080100	PST 500	108530361
4080100	PST 500	108530362
4080100	PST 500	108530363
4080100	PST 500	108530364
4080100	PST 500	108530365
4080100	PST 500	108546969

Attachment A: Affected Product Table

4080100	PST 500	108547029
4080100	PST 500	108547031
4080100	PST 500	108547032
4080100	PST 500	108547033
4080100	PST 500	108547035
4080100	PST 500	108547042
4080100	PST 500	108547043
4080100	PST 500	108547044
4080100	PST 500	108547046
4080100	PST 500	108547051
4080100	PST 500	108547079
4080100	PST 500	108547136
4080100	PST 500	108547168
4080100	PST 500	108547176
4080100	PST 500	108547178
4080100	PST 500	108547185
4080100	PST 500	108547186
4080100	PST 500	108701335
4080100	PST 500	108850252
4080100	PST 500	108850253
4080100	PST 500	108850256
4080100	PST 500	108850259
4080100	PST 500	108850261
4080100	PST 500	108850265
4080100	PST 500	108850266
4080100	PST 500	108865826
4080100	PST 500	108865827
4080100	PST 500	108865828
4080100	PST 500	108865829
4080100	PST 500	108865830
4080100	PST 500	108865831
4080100	PST 500	108865832
4080100	PST 500	108865833
4080100	PST 500	108865834
4080100	PST 500	109213575
4080100	PST 500	109213578
4080100	PST 500	109297546
4080100	PST 500	109297547
4080100	PST 500	109297548
4080100	PST 500	109297549
4080100	PST 500	109401141

Attachment A: Affected Product Table

4080100	PST 500	109401142
4080100	PST 500	109401145
4080100	PST 500	109401146
4080100	PST 500	109405704
4080100	PST 500	109405711
4080100	PST 500	109427276
4080100	PST 500	109445771
4080100	PST 500	109445772
4080100	PST 500	109465769
4080100	PST 500	108275108
4080100	PST 500	108275109
4080100	PST 500	108288835
4080100	PST 500	108288836
4080100	PST 500	108288918
4080100	PST 500	108288922
4080100	PST 500	108288923
4080100	PST 500	108288927
4080100	PST 500	108486928
4080100	PST 500	108486929
4080100	PST 500	108501141
4080100	PST 500	109113369
4080100	PST 500	109180575
4080100	PST 500	109180576
4080100	PST 500	109180592
4080100	PST 500	108597121
4080100	PST 500	108597122
4080100	PST 500	108233125
4080100	PST 500	108233127
4080100	PST 500	108233128
4080100	PST 500	108233130
4080100	PST 500	108339137
4080100	PST 500	108339138
4080100	PST 500	108339139
4080100	PST 500	108339140
4080100	PST 500	108339141
4080100	PST 500	108339142
4080100	PST 500	108339143
4080100	PST 500	108339144
4080100	PST 500	108339145
4080100	PST 500	108339146
4080100	PST 500	108339147

Attachment A: Affected Product Table

4080100	PST 500	108339148
4080100	PST 500	108339149
4080100	PST 500	108339150
4080100	PST 500	108339151
4080100	PST 500	108978899
4080100	PST 500	108978900
4080100	PST 500	108978901
4080100	PST 500	108978902
4080100	PST 500	109072427
4080100	PST 500	109072428
4080100	PST 500	109072436
4080100	PST 500	109072437
4080100	PST 500	109072455
4080100	PST 500	109072480
4080100	PST 500	109072481
4080100	PST 500	109072484
4080100	PST 500	109072488
4080100	PST 500	109072489
4080100	PST 500	109072492
4080100	PST 500	109171331
4080100	PST 500	109171332
4080100	PST 500	109189120
4080100	PST 500	109189121
4080100	PST 500	109189127
4080100	PST 500	109189133
4080100	PST 500	109189140
4080100	PST 500	109200067
4080100	PST 500	109200068
4080100	PST 500	109200071
4080100	PST 500	109200072
4080100	PST 500	109200075
4080100	PST 500	109200076
4080100	PST 500	109316688
4080100	PST 500	109316693
4080100	PST 500	109316696
4080100	PST 500	109486022
4080100	PST 500	109486023
4080100	PST 500	109486026
4080100	PST 500	109486027
4080100	PST 500	109486029
4080100	PST 500	109500363

Attachment A: Affected Product Table

4080100	PST 500	109500464
4080100	PST 500	108160151
4080100	PST 500	108241935
4080100	PST 500	108424300
4080100	PST 500	108501140
4080100	PST 500	108634386
4080100	PST 500	108634387
4080100	PST 500	108716148
4080100	PST 500	108850240
4080100	PST 500	108850241
4080100	PST 500	108850250
4080100	PST 500	109004467
4080100	PST 500	108381286
4080100	PST 500	108381541
4080100	PST 500	108381543
4080100	PST 500	108381544
4080100	PST 500	108381546
4080100	PST 500	108381547
4080100	PST 500	108381548
4080100	PST 500	108381549
4080100	PST 500	108381550
4080100	PST 500	108381551
4080100	PST 500	108381552
4080100	PST 500	108392477
4080100	PST 500	108392478
4080100	PST 500	108492496
4080100	PST 500	108492517
4080100	PST 500	108501138
4080100	PST 500	108650849
4080100	PST 500	108650850
4080100	PST 500	108683203
4080100	PST 500	108850226
4080100	PST 500	108850229
4080100	PST 500	109218569
4080100	PST 500	109218570
4080100	PST 500	109218571
4080100	PST 500	109218572
4080100	PST 500	109218579
4080100	PST 500	109229484
4080100	PST 500	109309707
4080100	PST 500	109309708

Attachment A: Affected Product Table

4080100	PST 500	109309712
4080100	PST 500	109309713
4080100	PST 500	108160135
4080100	PST 500	108247824
4080100	PST 500	108247825
4080100	PST 500	108302860
4080100	PST 500	108492518
4080100	PST 500	108547191
4080100	PST 500	108570113
4080100	PST 500	108570623
4080100	PST 500	108597113
4080100	PST 500	108649030
4080100	PST 500	108672924
4080100	PST 500	108770719
4080100	PST 500	108850244
4080100	PST 500	108850246
4080100	PST 500	108850249
4080100	PST 500	108850251
4080100	PST 500	108877079
4080100	PST 500	108901260
4080100	PST 500	108903848
4080100	PST 500	109030699
4080100	PST 500	109030700
4080100	PST 500	109031470
4080100	PST 500	109031473
4080100	PST 500	109041036
4080100	PST 500	109041065
4080100	PST 500	109061367
4080100	PST 500	109218612
4080100	PST 500	109233194
4080100	PST 500	109233195
4080100	PST 500	109233196
4080100	PST 500	109233197
4080100	PST 500	109297543
4080100	PST 500	109297544
4080100	PST 500	109299627
4080100	PST 500	109308173
4080100	PST 500	109315026
4080100	PST 500	109315031
4080100	PST 500	109328463
4080100	PST 500	109328482

Attachment A: Affected Product Table

4080100	PST 500	109502053
4080100	PST 500	108241936
4080100	PST 500	108781141
4080100	PST 500	109229488
4080100	PST 500	109229489
4080100	PST 500	109229490
4080100	PST 500	109397454
4080100	PST 500	109397455
4080100	PST 500	108310309
4080100	PST 500	108310312
4080100	PST 500	108781163
4080100	PST 500	109377969
4080100	PST 500	109377977
4080100	PST 500	109381324
4080100	PST 500	109381325
4080100	PST 500	108058061
4080100	PST 500	108058108
4080100	PST 500	108058109
4080100	PST 500	108058110
4080100	PST 500	108058111
4080100	PST 500	108059601
4080100	PST 500	108059617
4080100	PST 500	108338469
4080100	PST 500	108339135
4080100	PST 500	108776160
4080100	PST 500	108779753
4080100	PST 500	108779903
4080100	PST 500	108780210
4080100	PST 500	108780213
4080100	PST 500	108780373
4080100	PST 500	108780379
4080100	PST 500	108780393
4080100	PST 500	108780510
4080100	PST 500	108780513
4080100	PST 500	109364905
4080100	PST 500	108241932
4080100	PST 500	108294089
4080100	PST 500	108597114
4080100	PST 500	108865837
4080100	PST 500	108865838
4080100	PST 500	108865839

Attachment A: Affected Product Table

4080100	PST 500	108865840
4080100	PST 500	108865841
4080100	PST 500	108865842
4080100	PST 500	108865843
4080100	PST 500	108865844
4080100	PST 500	108922935
4080100	PST 500	108965790
4080100	PST 500	108965791
4080100	PST 500	108965792
4080100	PST 500	108965793
4080100	PST 500	108965794
4080100	PST 500	108976300
4080100	PST 500	108976301
4080100	PST 500	108976302
4080100	PST 500	108976303
4080100	PST 500	108977024
4080100	PST 500	108977025
4080100	PST 500	108977026
4080100	PST 500	108977027
4080100	PST 500	108977031
4080100	PST 500	108977035
4080100	PST 500	109233206
4080100	PST 500	109243533
4080100	PST 500	109243536
4080100	PST 500	109243537
4080100	PST 500	109243554
4080100	PST 500	109299613
4080100	PST 500	109299614
4080100	PST 500	109299616
4080100	PST 500	109328476
4080100	PST 500	109328481
4080100	PST 500	109349072
4080100	PST 500	109349073
4080100	PST 500	108338481
4080100	PST 500	108570462
4080100	PST 500	108697607
4080100	PST 500	108339163
4080100	PST 500	108339165
4080100	PST 500	108370159
4080100	PST 500	108683204
4080100	PST 500	108757445

Attachment A: Affected Product Table

4080100	PST 500	108789747
4080100	PST 500	108833982
4080100	PST 500	108833985
4080100	PST 500	108893833
4080100	PST 500	108895123
4080100	PST 500	108895124
4080100	PST 500	108895125
4080100	PST 500	108895176
4080100	PST 500	108993453
4080100	PST 500	108993454
4080100	PST 500	108993455
4080100	PST 500	108993458
4080100	PST 500	108993459
4080100	PST 500	108993460
4080100	PST 500	108993461
4080100	PST 500	108993462
4080100	PST 500	108993465
4080100	PST 500	108993470
4080100	PST 500	108993473
4080100	PST 500	108993530
4080100	PST 500	108993534
4080100	PST 500	108993542
4080100	PST 500	108993543
4080100	PST 500	108993544
4080100	PST 500	109397431
4080100	PST 500	109397432
4080100	PST 500	108825608
4080100	PST 500	109113379
4080100	PST 500	109254356
4080100	PST 500	109254360
4080100	PST 500	109254362
4080100	PST 500	109254363
4080100	PST 500	109254364
4080100	PST 500	109254367
4080100	PST 500	109254368
4080100	PST 500	109377965
4080100	PST 500	109465608
4080100	PST 500	108339162
4080100	PST 500	108339166
4080100	PST 500	108391589
4080100	PST 500	108640144

Attachment A: Affected Product Table

4080100	PST 500	108469421
4080100	PST 500	108469424
4080100	PST 500	108469425
4080100	PST 500	108463472