
To: Distributors, Hospitals, Physicians

Field Safety Notice

Affected Product: Figulla Flex II ASD and its Procedure Pack

REF: 29ASD09F, 29ASD09P, 29ASD09FC, 29ASD09PC,
29ASD10F, 29ASD10P, 29ASD10FC, 29ASD10PC

FSN Identifier: FSN-2024-002

Notification of Device and Delivery system compatibility

Attention:

Occlutech is providing updated guidance on delivery system compatibility. Certain Figulla Flex II ASD and its Procedure Pack products have a size recommendation of Occlutech Delivery Set III (ODS III) that may lead to incompatibility.

Details on affected devices:

Product name: **Figulla Flex II ASD and its Procedure Pack**

Please refer to *Attachment FSN-2024-002_List of affected products*.

Dear Customer,

This Field Safety Notice is to inform you that Occlutech has initiated a voluntary Field Safety Notice for the products listed on the *Attachment FSN-2024-002_List of affected products*. We ask that you take note of the recommendations as described below. This Field Safety Notice is limited to the serial numbers mentioned on the list attached to this Field Safety Notice.

Description of the problem:

Occlutech has identified that products affected by this corrective action have a size recommendation of ODS III (REF 98DS006) on the label of the Procedure Pack box and the sterile pouch of the implant that might lead to incompatibility.

The recommendation and labelling of the Occlutech Delivery Set with REF 51DS007 as 7F on the affected Procedure Pack box and the sterile pouch of the affected implant remains correct as per specification.

It was identified that there were some discrepancies between the labelled size for ODS III and the results from internal tests.

The size of compatible delivery system on the labels is too small. Instead of the labelled 6F ODS III size, a minimum size of 7F ODS III should be used. The usage of an undersized delivery system for the implantation of the Figulla Flex II ASD could result in damage to the Occluder device.



Occlutech International AB • Landskronavägen 2 • SE-252 32 Helsingborg • Sweden

FSN identifier: FSN-2024-002
FSCA identifier: FSCA-2024-002

Date: 2024-12-11

To date, there have been two product complaints associated with this incorrect labelling. No patient adverse events or injury have been reported.

Previously distributed marketing materials, reporting ODSIII incorrect sizes, shall be considered outdated and not valid. Updated materials will be distributed as soon as they will be available.

Actions to be taken by the distributors, physicians and/or hospitals:

1. Inform clinicians and users of the updated recommendation regarding the recommendation of using 7F Occlutech Delivery Set III - ODS III (REF 98DS007) instead of 6F Occlutech Delivery Set III - ODS III (REF 98DS006) with Figulla Flex II ASD size 9 and 10.
2. A written confirmation of the receipt, the reading and understanding of the Field Safety Notice should be provided to info.mds@occlutech.com. Please also refer to the contact information mentioned at the end of this letter. If you have any questions or would like assistance with this Field Safety Notice, please contact your local Sales Representative.

Recommendation

1. Either the distributor or hospital should attach a copy of this Field Safety Notice to the outer box of any affected devices to act as a reminder.



Occlutech International AB • Landskronavägen 2 • SE-252 32 Helsingborg • Sweden

FSN identifier: FSN-2024-002
FSCA identifier: FSCA-2024-002

Date: 2024-12-11

Transmission of this Field Safety Notice (if appropriate):

This notice needs to be shared with users (physicians and/or hospitals) of the affected devices. Please assure and maintain awareness of this FSN among users of the affected Figulla Flex II ASD and its Procedure Pack products to ensure proper use of products.

Action to be taken by Occlutech:

Occlutech has taken all necessary steps to notify relevant authorities and has implement corrective and preventive actions.



Occlutech International AB • Landskronavägen 2 • SE-252 32 Helsingborg • Sweden

FSN identifier: FSN-2024-002
FSCA identifier: FSCA-2024-002

Date: 2024-12-11

Contact reference Customer Service:

Occlutech International AB
Landskronavägen 2
SE-252 32 Helsingborg,
Sweden

Contact Email address: order@occlutech.com
Contact Phone Number: +46 76636 7200

Medical Device Safety Officer
E- mail: info.mds@occlutech.com

Please do not hesitate to contact Occlutech with question concerning this FSN.

We apologize for these inconveniences and are working to resolve the issue as soon as possible. We trust that this document clarifies all concerns.

Dr. Morwan Choli
Managing Director, Occlutech GmbH
Executive Vice President RA&Q

Procedure pack description	Procedure pack REF	Serial Number Implant	Expire Date
Figulla Flex II ASD 9; Flex-Pusher II	29ASD09F	2331290909_01	2028-10-03
	29ASD09F	2331290909_03	2028-10-03
	29ASD09F	2331290909_05	2028-10-03
	29ASD09F	2331290909_10	2028-10-03
	29ASD09F	2331290909_12	2028-10-03
	29ASD09F	2331290909_14	2028-10-03
	29ASD09F	2331290909_21	2028-10-03
	29ASD09F	2331290909_25	2028-10-03
	29ASD09F	2331290909_27	2028-10-03
	29ASD09F	2331290909_28	2028-10-03
	29ASD09F	2331290909_29	2028-10-03
	29ASD09F	2343290916_02	2028-10-22
	29ASD09F	2343290916_03	2028-10-22
	29ASD09F	2343290916_06	2028-10-22
	29ASD09F	2343290916_15	2028-10-22
	29ASD09F	2343290916_16	2028-10-22
	29ASD09F	2343290916_17	2028-10-22
	29ASD09F	2343290916_18	2028-10-22
	29ASD09F	2343290916_20	2028-10-22
	29ASD09F	2343290916_23	2028-10-22
	29ASD09F	2343290916_25	2028-10-22
	29ASD09F	2343290916_27	2028-10-22
	29ASD09F	2343290916_29	2028-10-22
	29ASD09F	2344290901_02	2028-10-03
	29ASD09F	2344290901_03	2028-10-03
	29ASD09F	2344290901_04	2028-10-03
	29ASD09F	2344290901_05	2028-10-03
	29ASD09F	2344290901_06	2028-10-03
	29ASD09F	2344290901_07	2028-10-03
	29ASD09F	2344290901_09	2028-10-03
	29ASD09F	2344290901_10	2028-10-03
	29ASD09F	2344290901_11	2028-10-03
	29ASD09F	2344290901_12	2028-10-03
	29ASD09F	2344290901_13	2028-10-03
	29ASD09F	2344290901_14	2028-10-03
	29ASD09F	2344290901_15	2028-10-03
	29ASD09F	2344290901_17	2028-10-03
	29ASD09F	2344290901_19	2028-10-03
	29ASD09F	2344290901_21	2028-10-03
	29ASD09F	2344290901_22	2028-10-03
	29ASD09F	2344290901_24	2028-10-03
	29ASD09F	2344290901_25	2028-10-03
	29ASD09F	2344290901_26	2028-10-03
	29ASD09F	2344290901_29	2028-10-03
29ASD09F	2344290901_30	2028-10-03	
29ASD09F	2346290914_30	2029-01-23	
29ASD09F	2403290901_01	2029-02-12	
29ASD09F	2403290901_02	2029-02-12	
29ASD09F	2403290901_03	2029-02-12	
29ASD09F	2403290901_05	2029-02-12	
29ASD09F	2403290901_06	2029-02-12	
29ASD09F	2403290901_07	2029-02-12	
29ASD09F	2403290901_08	2029-02-12	
29ASD09F	2403290901_10	2029-02-12	



Procedure pack description	Procedure pack REF	Serial Number Implant	Expire Date
	29ASD09F	2403290901_11	2029-02-12
	29ASD09F	2403290901_12	2029-02-12
	29ASD09F	2403290901_13	2027-03-14
	29ASD09F	2403290901_15	2027-03-14
	29ASD09F	2403290901_16	2029-02-14
	29ASD09F	2403290901_19	2029-02-14
	29ASD09F	2403290901_20	2029-02-14
	29ASD09F	2403290901_21	2029-02-14
	29ASD09F	2403290901_22	2029-02-14
	29ASD09F	2403290901_23	2029-02-14
	29ASD09F	2403290901_24	2029-02-14
	29ASD09F	2403290901_25	2027-09-27
	29ASD09F	2403290901_27	2027-09-27
	29ASD09F	2403290901_28	2027-09-27
	29ASD09F	2412290916_02	2029-04-09
	29ASD09F	2412290916_04	2029-04-09
	29ASD09F	2412290916_05	2029-04-09
	29ASD09F	2412290916_07	2029-04-09
	29ASD09F	2412290916_09	2029-04-09
	29ASD09F	2412290916_14	2029-04-09
	29ASD09F	2412290916_16	2029-04-09
	29ASD09F	2412290916_17	2029-04-09
	29ASD09F	2412290916_18	2029-04-09
	29ASD09F	2412290916_20	2029-04-09
	29ASD09F	2412290916_22	2029-04-09
	29ASD09F	2412290916_23	2029-04-09
	29ASD09F	2412290916_24	2029-04-09
	29ASD09F	2412290916_25	2029-04-09
	29ASD09F	2412290916_27	2029-04-09
	29ASD09F	2412290916_28	2029-04-09
	29ASD09F	2412290916_29	2029-04-09
	29ASD09F	2412290916_30	2029-04-09
	29ASD09F	2419290908_08	2029-04-17
	29ASD09F	2419290908_11	2029-04-17
	29ASD09F	2419290908_14	2029-04-17
	29ASD09F	2419290908_15	2029-04-17
	29ASD09F	2419290908_16	2029-04-17
	29ASD09F	2419290908_30	2029-04-17
	29ASD09F	2425290904_03	2027-06-21
	29ASD09F	2425290904_04	2027-06-21
	29ASD09F	2425290904_05	2027-06-21
	29ASD09F	2425290904_06	2027-01-09
	29ASD09F	2432290904_01	2029-05-07
	29ASD09F	2432290904_02	2029-05-07
Figulla Flex II ASD 9; Occlutech Pistol Pusher	29ASD09P	2346290914_01	2029-01-23
	29ASD09P	2346290914_02	2029-01-23
	29ASD09P	2346290914_03	2029-01-23
	29ASD09P	2346290914_04	2029-01-23
	29ASD09P	2346290914_05	2029-01-23
	29ASD09P	2346290914_06	2029-01-23
	29ASD09P	2346290914_07	2029-01-23
	29ASD09P	2346290914_10	2029-01-23
	29ASD09P	2346290914_11	2029-01-23
	29ASD09P	2346290914_12	2029-01-23



Procedure pack description	Procedure pack REF	Serial Number Implant	Expire Date
	29ASD09P	2346290914_13	2029-01-23
	29ASD09P	2346290914_14	2029-01-23
	29ASD09P	2346290914_15	2029-01-23
	29ASD09P	2346290914_16	2029-01-23
	29ASD09P	2346290914_17	2029-01-23
	29ASD09P	2346290914_18	2029-01-23
	29ASD09P	2346290914_20	2029-01-23
	29ASD09P	2346290914_21	2029-01-23
	29ASD09P	2346290914_22	2029-01-23
	29ASD09P	2346290914_23	2029-01-23
	29ASD09P	2346290914_24	2029-01-23
	29ASD09P	2346290914_26	2029-01-23
	29ASD09P	2346290914_27	2029-01-23
	29ASD09P	2346290914_28	2029-01-23
	29ASD09P	2346290914_29	2029-01-23
	29ASD09P	2414290913_01	2029-03-17
	29ASD09P	2414290913_02	2029-03-17
	29ASD09P	2414290913_03	2029-03-17
	29ASD09P	2414290913_04	2029-03-17
	29ASD09P	2414290913_07	2029-03-17
	29ASD09P	2414290913_08	2029-03-17
	29ASD09P	2414290913_09	2029-03-17
	29ASD09P	2414290913_12	2029-03-17
	29ASD09P	2414290913_17	2029-03-17
	29ASD09P	2414290913_19	2029-03-17
	29ASD09P	2414290913_30	2029-03-17
	29ASD09P	2423290905_01	2029-04-29
	29ASD09P	2423290905_02	2029-04-29
	29ASD09P	2423290905_08	2029-04-29
	29ASD09P	2423290905_09	2029-04-29
	29ASD09P	2423290905_11	2029-04-29
	29ASD09P	2423290905_12	2029-04-29
	29ASD09P	2423290905_13	2029-04-29
	29ASD09P	2423290905_14	2029-04-29
	29ASD09P	2423290905_16	2029-04-29
	29ASD09P	2423290905_17	2029-04-29
	29ASD09P	2423290905_18	2029-04-29
	29ASD09P	2423290905_19	2029-04-29
	29ASD09P	2423290905_20	2029-04-29
	29ASD09P	2423290905_23	2029-04-29
	29ASD09P	2423290906_04	2029-04-29
	29ASD09P	2423290906_05	2029-04-29
	29ASD09P	2423290906_06	2029-04-29
	29ASD09P	2423290906_07	2029-04-29
	29ASD09P	2423290906_08	2029-04-29
Figulla Flex II ASD 10,5; Flex-Pusher II	29ASD10F	2331291011_04	2028-05-02
	29ASD10F	2331291011_05	2028-05-02
	29ASD10F	2331291011_11	2028-05-02
	29ASD10F	2331291011_13	2028-05-02
	29ASD10F	2331291011_15	2028-05-02
	29ASD10F	2331291011_21	2028-05-02
	29ASD10F	2331291011_29	2028-05-02
	29ASD10F	2334291014_11	2028-11-01
	29ASD10F	2334291014_16	2028-11-01

Procedure pack description	Procedure pack REF	Serial Number Implant	Expire Date
	29ASD10F	2334291014_17	2028-11-01
	29ASD10F	2334291014_19	2028-11-01
	29ASD10F	2334291014_20	2028-11-01
	29ASD10F	2334291014_21	2028-11-01
	29ASD10F	2334291014_24	2028-11-01
	29ASD10F	2334291014_25	2028-11-01
	29ASD10F	2334291014_27	2028-11-01
	29ASD10F	2334291014_28	2028-11-01
	29ASD10F	2334291014_29	2028-11-01
	29ASD10F	2334291015_03	2028-05-02
	29ASD10F	2334291015_06	2028-05-02
	29ASD10F	2334291015_09	2028-05-02
	29ASD10F	2334291015_10	2028-05-02
	29ASD10F	2334291015_11	2028-05-02
	29ASD10F	2334291015_16	2028-05-02
	29ASD10F	2334291015_17	2028-05-02
	29ASD10F	2334291015_24	2028-05-02
	29ASD10F	2334291015_25	2028-05-02
	29ASD10F	2334291015_26	2028-05-02
	29ASD10F	2336291004_02	2028-10-03
	29ASD10F	2336291004_04	2028-10-03
	29ASD10F	2336291004_05	2028-10-03
	29ASD10F	2336291004_06	2028-10-03
	29ASD10F	2336291004_07	2028-10-03
	29ASD10F	2336291004_09	2028-10-03
	29ASD10F	2336291004_10	2028-10-03
	29ASD10F	2336291004_12	2028-10-03
	29ASD10F	2336291004_13	2028-10-03
	29ASD10F	2336291004_19	2028-10-03
	29ASD10F	2336291004_20	2028-10-03
	29ASD10F	2336291004_21	2028-10-03
	29ASD10F	2336291004_24	2028-10-03
	29ASD10F	2336291004_26	2028-10-03
	29ASD10F	2336291004_27	2028-10-03
	29ASD10F	2336291004_28	2028-10-03
	29ASD10F	2336291005_02	2028-09-19
	29ASD10F	2336291005_04	2028-09-19
	29ASD10F	2336291005_05	2028-09-19
	29ASD10F	2336291005_08	2028-09-19
	29ASD10F	2336291005_09	2028-09-19
	29ASD10F	2336291005_10	2028-09-19
	29ASD10F	2336291005_11	2028-09-19
	29ASD10F	2336291005_12	2028-09-19
	29ASD10F	2336291005_14	2028-09-19
	29ASD10F	2336291005_15	2028-09-19
	29ASD10F	2336291005_16	2028-09-19
	29ASD10F	2336291005_18	2028-09-19
	29ASD10F	2336291005_21	2028-09-19
	29ASD10F	2336291005_22	2028-09-19
	29ASD10F	2336291005_23	2028-09-19
	29ASD10F	2336291005_26	2028-09-19
	29ASD10F	2336291005_27	2028-09-19
	29ASD10F	2336291005_28	2028-09-19
	29ASD10F	2336291005_29	2028-09-19

Procedure pack description	Procedure pack REF	Serial Number Implant	Expire Date
	29ASD10F	2336291005_30	2028-09-19
	29ASD10F	2336291006_02	2028-05-02
	29ASD10F	2336291006_04	2028-05-02
	29ASD10F	2336291006_05	2028-05-02
	29ASD10F	2336291006_07	2028-05-02
	29ASD10F	2336291006_08	2028-05-02
	29ASD10F	2336291006_10	2028-05-02
	29ASD10F	2336291006_11	2028-05-02
	29ASD10F	2336291006_12	2028-05-02
	29ASD10F	2336291006_13	2028-05-02
	29ASD10F	2336291006_14	2028-05-02
	29ASD10F	2336291006_15	2028-05-02
	29ASD10F	2336291006_18	2028-05-02
	29ASD10F	2336291006_19	2028-05-02
	29ASD10F	2336291006_20	2028-05-02
	29ASD10F	2336291006_26	2028-05-02
	29ASD10F	2336291006_28	2028-05-02
	29ASD10F	2336291007_03	2028-09-19
	29ASD10F	2336291007_04	2028-09-19
	29ASD10F	2336291007_05	2028-09-19
	29ASD10F	2336291007_06	2028-09-19
	29ASD10F	2336291007_07	2028-09-19
	29ASD10F	2336291007_08	2028-09-19
	29ASD10F	2336291007_09	2028-09-19
	29ASD10F	2336291007_11	2028-09-19
	29ASD10F	2336291007_12	2028-09-19
	29ASD10F	2336291007_13	2028-09-19
	29ASD10F	2336291007_15	2028-09-19
	29ASD10F	2336291007_16	2028-09-19
	29ASD10F	2336291007_21	2028-09-19
	29ASD10F	2336291007_22	2028-09-19
	29ASD10F	2336291007_23	2028-09-19
	29ASD10F	2336291007_24	2028-09-19
	29ASD10F	2336291007_27	2028-09-19
	29ASD10F	2336291007_28	2028-09-19
	29ASD10F	2336291007_30	2028-09-19
	29ASD10F	2344291002_01	2028-10-03
	29ASD10F	2344291002_02	2028-10-03
	29ASD10F	2344291002_05	2028-10-03
	29ASD10F	2344291002_07	2028-10-03
	29ASD10F	2344291002_08	2028-10-03
	29ASD10F	2344291002_12	2028-10-03
	29ASD10F	2344291002_17	2028-10-03
	29ASD10F	2344291002_18	2028-10-03
	29ASD10F	2344291002_20	2028-10-03
	29ASD10F	2344291002_21	2028-10-03
	29ASD10F	2344291002_22	2028-10-03
	29ASD10F	2344291002_24	2027-03-21
	29ASD10F	2344291002_26	2027-03-21
	29ASD10F	2344291002_27	2027-03-21
	29ASD10F	2344291002_28	2027-03-21
	29ASD10F	2344291002_29	2027-03-21
	29ASD10F	2344291002_30	2027-03-21
	29ASD10F	2344291003_01	2028-10-03

Procedure pack description	Procedure pack REF	Serial Number Implant	Expire Date
	29ASD10F	2344291003_03	2028-10-03
	29ASD10F	2344291003_04	2028-10-03
	29ASD10F	2344291003_05	2028-10-03
	29ASD10F	2344291003_06	2028-10-03
	29ASD10F	2344291003_10	2028-10-03
	29ASD10F	2344291003_12	2028-10-03
	29ASD10F	2344291003_18	2028-10-03
	29ASD10F	2344291003_20	2028-10-03
	29ASD10F	2344291003_21	2028-10-03
	29ASD10F	2344291003_22	2028-10-03
	29ASD10F	2344291003_24	2028-10-03
	29ASD10F	2344291003_26	2028-10-03
	29ASD10F	2344291003_28	2028-10-03
	29ASD10F	2346291015_03	2028-10-22
	29ASD10F	2346291015_04	2028-10-22
	29ASD10F	2346291015_05	2028-10-22
	29ASD10F	2346291015_06	2028-10-22
	29ASD10F	2346291015_07	2028-10-22
	29ASD10F	2346291015_08	2028-10-22
	29ASD10F	2346291015_09	2028-10-22
	29ASD10F	2346291015_11	2028-10-22
	29ASD10F	2346291015_13	2028-10-22
	29ASD10F	2346291015_14	2028-10-22
	29ASD10F	2346291015_15	2028-10-22
	29ASD10F	2346291015_16	2028-10-22
	29ASD10F	2346291015_17	2028-10-22
	29ASD10F	2346291015_19	2028-10-22
	29ASD10F	2346291015_23	2028-10-22
	29ASD10F	2346291015_24	2028-10-22
	29ASD10F	2346291015_25	2028-10-22
	29ASD10F	2346291015_26	2028-10-22
	29ASD10F	2346291015_27	2028-10-22
	29ASD10F	2346291015_28	2028-10-22
	29ASD10F	2346291015_29	2028-10-22
	29ASD10F	2346291015_30	2028-10-22
	29ASD10F	2405291004_02	2029-02-12
	29ASD10F	2405291004_03	2029-02-12
	29ASD10F	2405291004_04	2029-02-12
	29ASD10F	2405291004_06	2029-02-12
	29ASD10F	2405291004_09	2029-02-12
	29ASD10F	2405291004_11	2029-02-12
	29ASD10F	2405291004_12	2029-02-12
	29ASD10F	2405291004_14	2029-02-12
	29ASD10F	2405291004_18	2029-02-12
	29ASD10F	2405291004_21	2029-02-12
	29ASD10F	2405291004_22	2029-02-12
	29ASD10F	2405291004_23	2029-02-12
	29ASD10F	2405291004_25	2029-02-12
	29ASD10F	2405291004_26	2029-02-12
	29ASD10F	2405291004_27	2029-02-12
	29ASD10F	2405291004_29	2029-02-12
	29ASD10F	2405291004_30	2029-02-12
	29ASD10F	2408291012_01	2029-04-17
	29ASD10F	2408291012_02	2029-04-17



Procedure pack description	Procedure pack REF	Serial Number Implant	Expire Date
	29ASD10F	2408291012_05	2029-04-17
	29ASD10F	2408291012_06	2029-04-17
	29ASD10F	2408291013_02	2029-02-12
	29ASD10F	2408291013_04	2029-02-12
	29ASD10F	2408291013_05	2029-02-12
	29ASD10F	2408291013_06	2029-02-12
	29ASD10F	2408291013_07	2029-02-12
	29ASD10F	2408291013_08	2029-02-12
	29ASD10F	2408291013_09	2029-02-12
	29ASD10F	2408291013_10	2029-02-12
	29ASD10F	2408291013_11	2029-02-12
	29ASD10F	2408291013_13	2029-02-12
	29ASD10F	2408291013_22	2029-01-15
	29ASD10F	2408291013_23	2029-01-15
	29ASD10F	2408291013_24	2029-01-15
	29ASD10F	2411291016_02	2026-06-12
	29ASD10F	2411291016_06	2026-06-12
	29ASD10F	2411291016_07	2026-06-12
	29ASD10F	2411291016_10	2026-06-12
	29ASD10F	2411291016_11	2026-06-12
	29ASD10F	2411291016_12	2026-05-14
	29ASD10F	2411291016_13	2026-05-14
	29ASD10F	2411291016_15	2026-05-14
	29ASD10F	2411291016_16	2026-05-14
	29ASD10F	2411291016_17	2026-05-14
	29ASD10F	2415291002_04	2029-04-09
	29ASD10F	2415291002_05	2029-04-09
	29ASD10F	2415291002_08	2029-04-09
	29ASD10F	2415291002_10	2029-04-09
	29ASD10F	2415291002_11	2029-04-09
	29ASD10F	2415291002_12	2029-04-09
	29ASD10F	2415291002_15	2029-04-09
	29ASD10F	2415291002_16	2029-04-09
	29ASD10F	2415291002_17	2029-04-09
	29ASD10F	2415291002_18	2029-04-09
	29ASD10F	2415291002_19	2029-04-09
	29ASD10F	2415291002_22	2029-04-09
	29ASD10F	2415291002_23	2029-04-09
	29ASD10F	2415291002_24	2029-04-09
	29ASD10F	2415291002_26	2029-04-09
	29ASD10F	2415291002_27	2029-04-09
	29ASD10F	2415291002_30	2029-04-09
	29ASD10F	2418291014_01	2029-04-17
	29ASD10F	2418291014_02	2029-04-17
	29ASD10F	2418291014_03	2029-04-17
	29ASD10F	2418291014_05	2029-04-17
	29ASD10F	2418291014_06	2029-04-17
	29ASD10F	2418291014_07	2029-04-17
	29ASD10F	2418291014_09	2029-04-17
	29ASD10F	2418291014_11	2029-04-17
	29ASD10F	2418291014_12	2029-04-17
	29ASD10F	2418291014_14	2029-04-17
	29ASD10F	2418291014_15	2029-04-17
	29ASD10F	2418291014_16	2029-04-17

Procedure pack description	Procedure pack REF	Serial Number Implant	Expire Date
	29ASD10F	2418291014_17	2029-04-17
	29ASD10F	2418291014_18	2029-04-17
	29ASD10F	2418291014_20	2029-04-17
	29ASD10F	2418291014_22	2029-04-17
	29ASD10F	2418291014_26	2029-04-17
	29ASD10F	2418291014_27	2029-04-17
	29ASD10F	2418291014_28	2029-04-17
	29ASD10F	2418291015_01	2029-05-07
	29ASD10F	2418291015_02	2029-05-07
	29ASD10F	2418291015_03	2029-05-07
	29ASD10F	2418291015_07	2029-05-07
	29ASD10F	2418291015_14	2029-05-07
	29ASD10F	2423291014_01	2029-05-13
	29ASD10F	2423291014_02	2029-05-13
	29ASD10F	2423291014_03	2029-05-13
	29ASD10F	2434291007_03	2029-04-29
	29ASD10F	2434291007_04	2029-04-29
	29ASD10F	2434291007_05	2029-04-29
	29ASD10F	2434291007_06	2029-04-29
	29ASD10F	2434291007_08	2029-04-29
	29ASD10F	2434291007_09	2029-04-29
	29ASD10F	2434291007_10	2029-04-29
	29ASD10F	2434291007_11	2029-04-29
	29ASD10F	2434291007_13	2029-04-29
	29ASD10F	2434291007_14	2029-04-29
	29ASD10F	2434291007_15	2029-04-29
	29ASD10F	2434291007_16	2029-04-29
	29ASD10F	2434291007_18	2029-04-29
	29ASD10F	2434291007_19	2029-04-29
	29ASD10F	2434291007_21	2029-04-29
	29ASD10F	2434291007_22	2029-04-29
	29ASD10F	2434291007_23	2029-04-29
	29ASD10F	2434291007_24	2029-04-29
	29ASD10F	2434291007_26	2029-04-29
	29ASD10F	2434291007_27	2029-04-29
	29ASD10F	2434291007_30	2029-04-29
Figulla Flex II ASD 10,5; Occlutech Pistol Pusher	29ASD10P	2408291013_26	2029-02-26
	29ASD10P	2408291013_18	2029-02-26
	29ASD10P	2403291002_28	2029-02-19
	29ASD10P	2403291002_29	2029-02-19
	29ASD10P	2403291002_27	2029-02-19
	29ASD10P	2411291015_18	2029-03-05
	29ASD10P	2411291015_17	2029-03-05
	29ASD10P	2411291015_16	2029-03-05
	29ASD10P	2427291022_29	2029-03-16
	29ASD10P	2408291013_29	2029-02-26
	29ASD10P	2411291015_05	2029-03-05
	29ASD10P	2415291003_13	2029-03-17
	29ASD10P	2411291015_24	2029-03-05
	29ASD10P	2411291015_29	2029-03-05
	29ASD10P	2415291003_15	2029-03-17
	29ASD10P	2411291015_28	2029-03-05
	29ASD10P	2415291003_08	2029-03-17
	29ASD10P	2415291003_07	2029-03-17

Procedure pack description	Procedure pack REF	Serial Number Implant	Expire Date
	29ASD10P	2415291003_17	2029-03-17
	29ASD10P	2415291003_06	2029-03-17
	29ASD10P	2403291002_20	2029-02-19
	29ASD10P	2403291002_05	2029-02-19
	29ASD10P	2403291002_03	2029-02-19
	29ASD10P	2411291015_07	2029-03-05
	29ASD10P	2411291015_06	2029-03-05
	29ASD10P	2427291022_21	2029-03-16
	29ASD10P	2427291022_20	2029-03-16
	29ASD10P	2411291015_21	2029-03-05
	29ASD10P	2415291003_12	2029-03-17
	29ASD10P	2415291003_11	2029-03-17
	29ASD10P	2411291015_12	2029-03-05
	29ASD10P	2411291015_01	2029-03-05
	29ASD10P	2427291022_09	2029-03-16
	29ASD10P	2415291003_28	2029-03-17
	29ASD10P	2408291013_19	2029-02-26
	29ASD10P	2411291015_10	2029-03-05
	29ASD10P	2411291015_08	2029-03-05
	29ASD10P	2403291002_21	2029-02-19
	29ASD10P	2408291013_28	2029-02-26
	29ASD10P	2408291013_27	2029-02-26
	29ASD10P	2408291013_16	2029-02-26
	29ASD10P	2403291002_17	2029-02-19
	29ASD10P	2403291002_13	2029-02-19
	29ASD10P	2403291002_08	2029-02-19
	29ASD10P	2427291022_19	2029-03-16
	29ASD10P	2427291022_18	2029-03-16
	29ASD10P	2427291022_17	2029-03-16
	29ASD10P	2427291022_14	2029-03-16
	29ASD10P	2427291022_13	2029-03-16
	29ASD10P	2427291022_27	2029-03-16
	29ASD10P	2403291002_24	2029-02-19
	29ASD10P	2411291015_19	2029-03-05
	29ASD10P	2427291022_12	2029-03-16
	29ASD10P	2411291016_27	2029-04-23
	29ASD10P	2415291003_20	2029-03-17
	29ASD10P	2411291015_25	2029-03-05
	29ASD10P	2411291015_11	2029-03-05
	29ASD10P	2403291002_26	2029-02-19
	29ASD10P	2403291002_25	2029-02-19
	29ASD10P	2411291015_13	2029-03-05
	29ASD10P	2403291002_18	2029-02-19
	29ASD10P	2411291015_22	2029-03-05
	29ASD10P	2408291013_15	2029-02-26
	29ASD10P	2408291013_14	2029-02-26
	29ASD10P	2408291013_21	2029-02-26
	29ASD10P	2408291013_20	2029-02-26
	29ASD10P	2411291016_26	2029-04-23
	29ASD10P	2411291016_19	2029-04-23
	29ASD10P	2415291003_18	2029-03-17
	29ASD10P	2411291015_15	2029-03-05
	29ASD10P	2411291015_14	2029-03-05
	29ASD10P	2403291002_01	2029-02-19

Procedure pack description	Procedure pack REF	Serial Number Implant	Expire Date
	29ASD10P	2427291022_11	2029-03-16
	29ASD10P	2427291022_10	2029-03-16
	29ASD10P	2408291013_30	2029-02-26
	29ASD10P	2427291022_24	2029-03-16
	29ASD10P	2427291022_22	2029-03-16
	29ASD10P	2403291002_23	2029-02-19
	29ASD10P	2411291016_25	2029-04-23